

# Quick Start Guide

# Lucerna™



Simulated  
Laser Beam

Power Laser Technology Powered by

**VIAX™**  
DENTAL TECHNOLOGIES



Take out the box with the Lucerna name.



Remove protective foam, then take out the unit from the box.



Remove the user manual and the handpiece from the box.



Remove the Accessories box.

Continue





Remove the battery unit from the accessories and check its charge level.



Install the battery inside the unit. If the battery is fully charged, there is no need to connect the power supply.



Press the main switch and wait 30 seconds for the Lucerna logo to appear.



Remove the charger from the Dental Laser Tips box.



If the battery has 2 bars or less, connect the charger. Do not use the unit with the charger attached.



Remove the protective cap from the handpiece and firmly insert all the way in the unit.



Remove screen protector. Press the password: 4-5-6.



Choose the Laser Tip and remove the plastic plugs, then thread on the handpiece.

Continue



13



Enjoy the use of your new laser and for additional inquiries visit:

**VIAx**<sup>TM</sup>

DENTAL TECHNOLOGIES

[www.viaxdental.com](http://www.viaxdental.com)